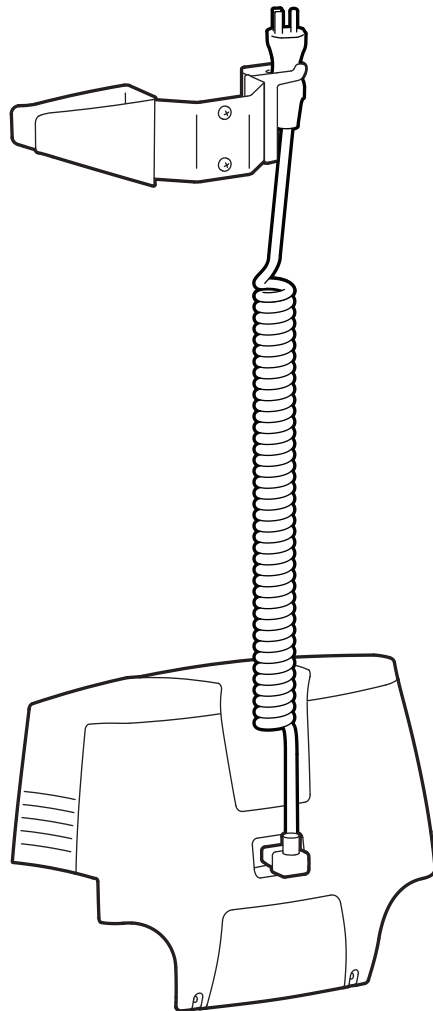
# ergotron®

---

## StyleView™ Power Cord Upgrade Kit for SLA Carts

---

**EN** User Guide

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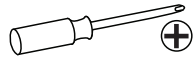
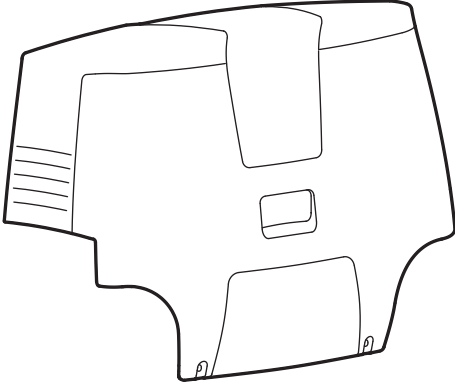
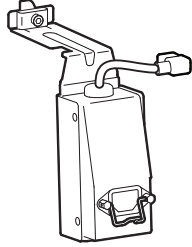
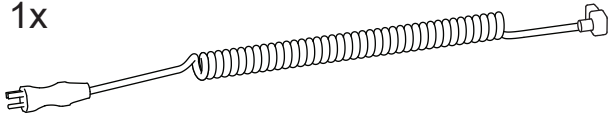
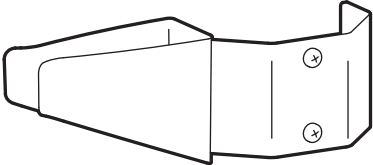
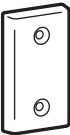
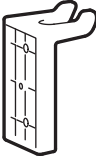
www.ergotron.com | USA: 1-800-888-8458 | Europe: +31 (0)33-45 45 600 | China: 400-120-3051 | Japan: japansupport@ergotron.com


---

Tools Needed

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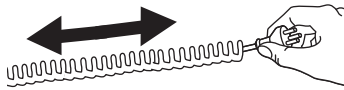
- EN** Do Not stretch coiled cord further than 6 feet (1.8 meters), damage to the cord may occur.
- ES** \*No alargue el cable espiral más de 1,8 metros (6 pies), ya que podría dañarlo.
- FR** Ne pas tirer sur le cordon sur plus de 6 pieds (1,8 mètres) pour éviter d'éventuels dommages.
- DE** Ziehen Sie das Spiralkabel nicht mehr als 1,8 Meter in die Länge, da es sonst zu Schäden am Kabel kommen kann.
- JA** コイル状コードは1.8メートル以上伸ばさないでください。コードが損傷する可能性があります。
- ZH** 不要将电源卷线拉伸至超过 1.8 米 (6 英尺) 长, 否则可导致电源线受损。
- NL** Trek het spiraalsnoer niet verder dan 1,8 meter uit, anders kan het snoer worden beschadigd.
- IT** Non tendere il cordone spiralato per una lunghezza superiore a 1,8 metri, altrimenti lo si può danneggiare.
- SV** Sträck inte ut spiralkabeln längre än 1,8 meter. I annat fall kan kabeln skadas.


1

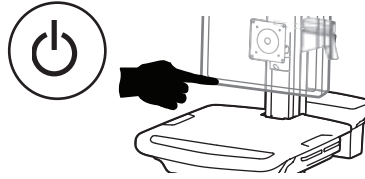Turn off all mounted equipment.

2

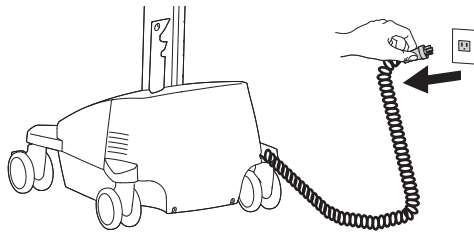Disconnect Power System from power source.

3

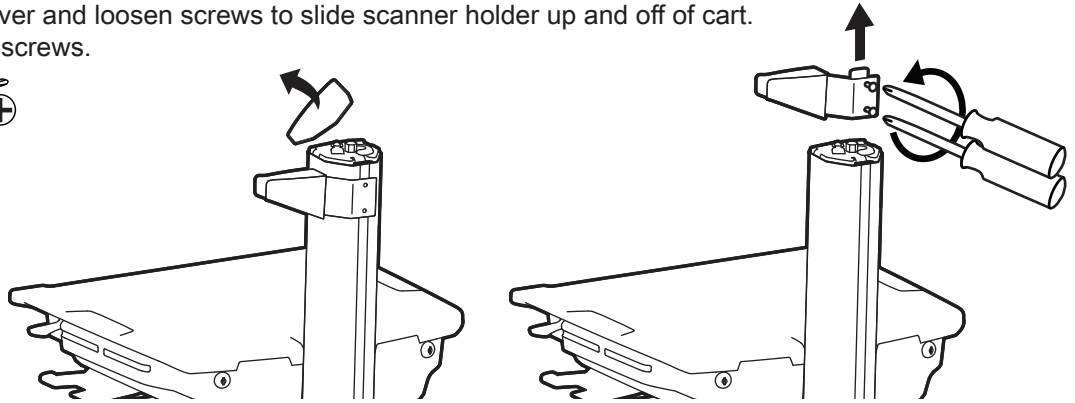
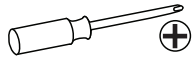
Turn power system off by holding down the AC Outlet Power button for 1 - 3 seconds. Power light will shut off.

- 100
- <90
- <70
- <50
- <30
- 



4

Remove top cover and loosen screws to slide scanner holder up and off of cart. Do not remove screws.



5

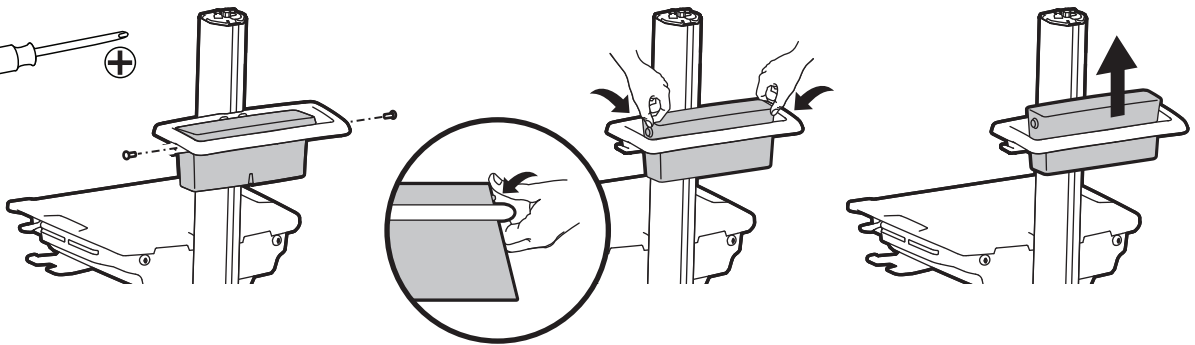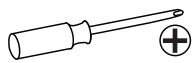Remove basket.



6

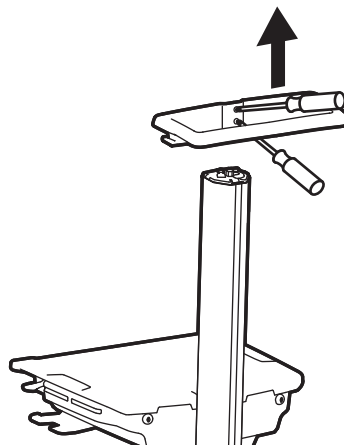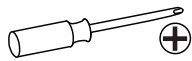Loosen screws to slide handle up and off of cart. Do not remove screws.


7

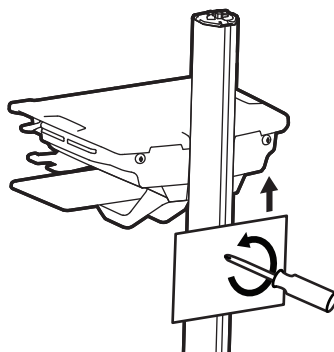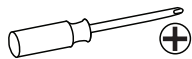Remove all other T-slot mounted equipment by loosening screws to slide up and off of cart. Do not remove screws.


# 8

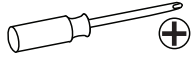
Remove front base cover.


1

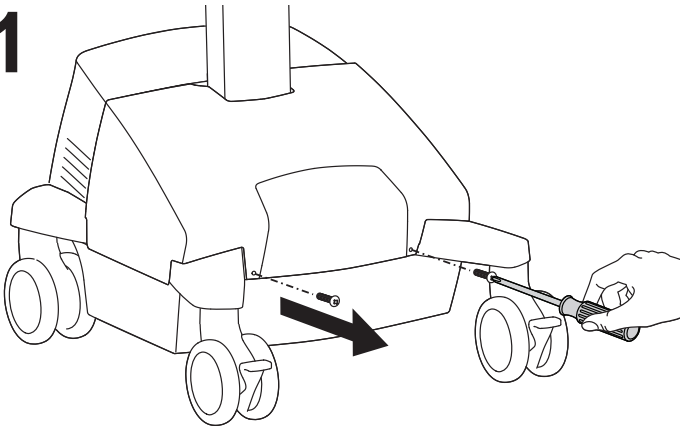2


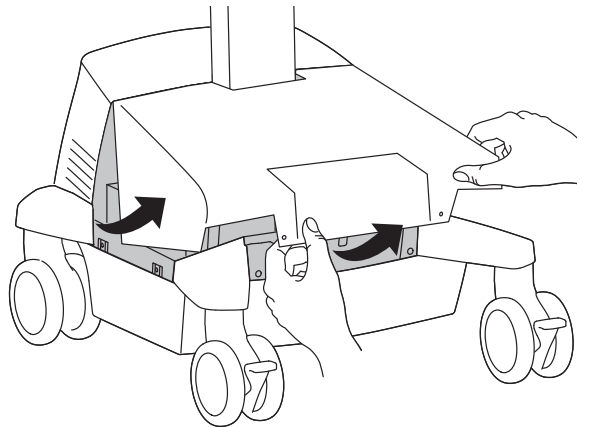
3


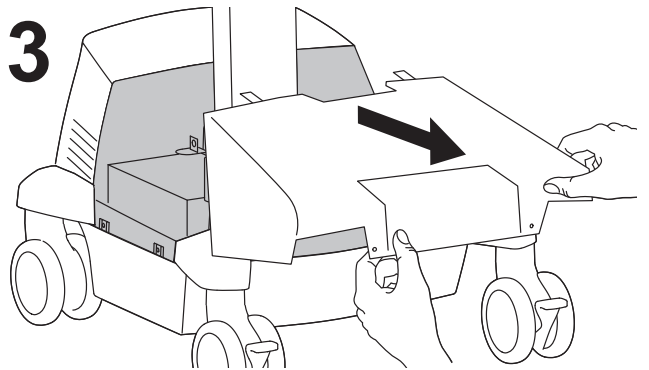
# 9

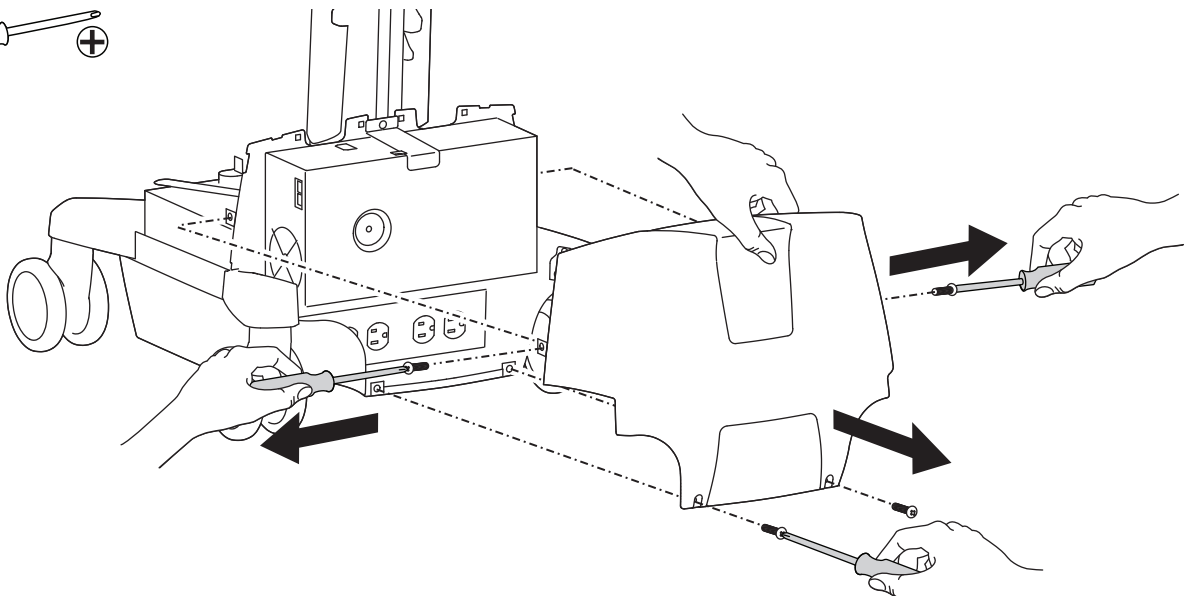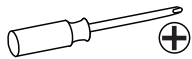Remove rear base cover.



**10**

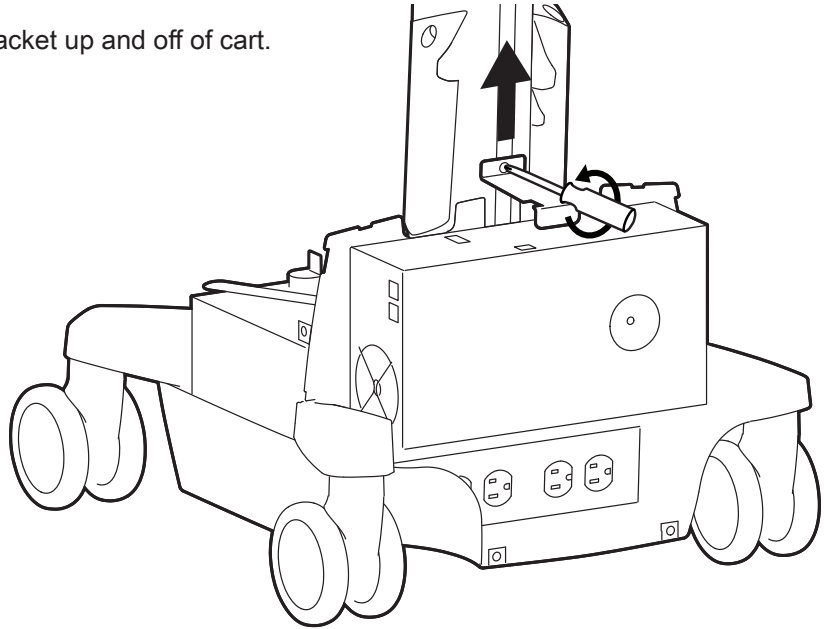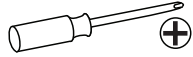Loosen screw to slide bracket up and off of cart.  
Do not remove screw.


**11**

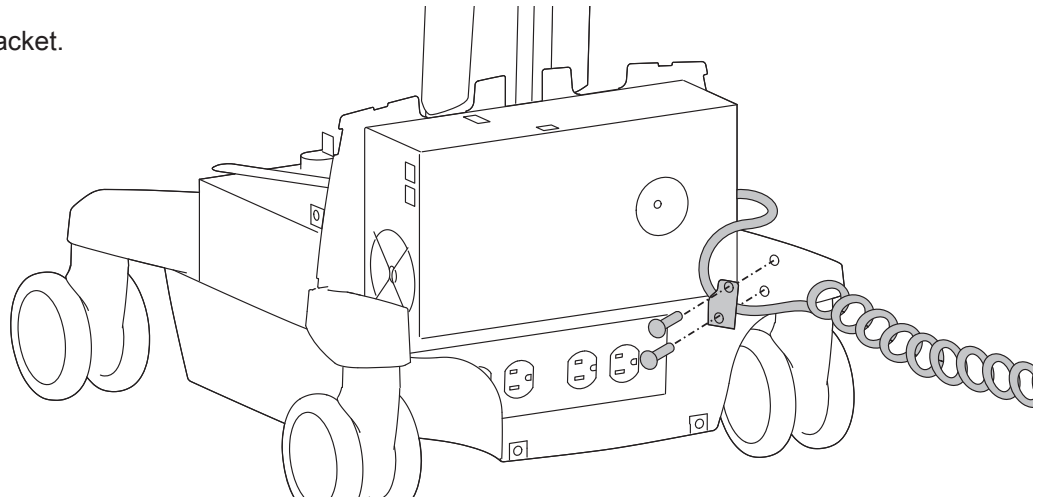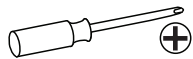Remove retainer bracket.

**12**

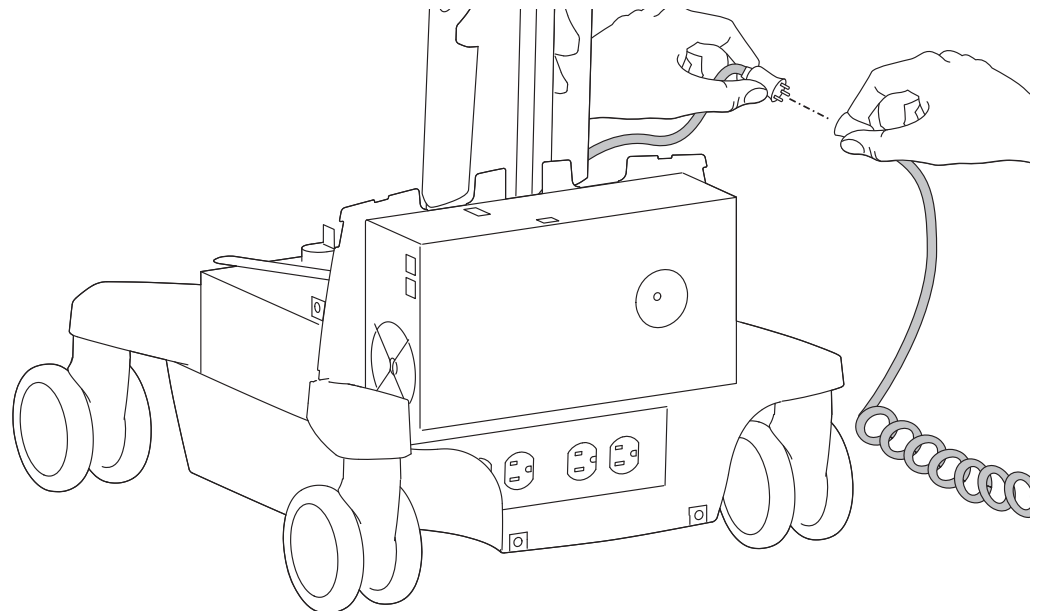
Unplug power cord.


# 13

Attach new power box.

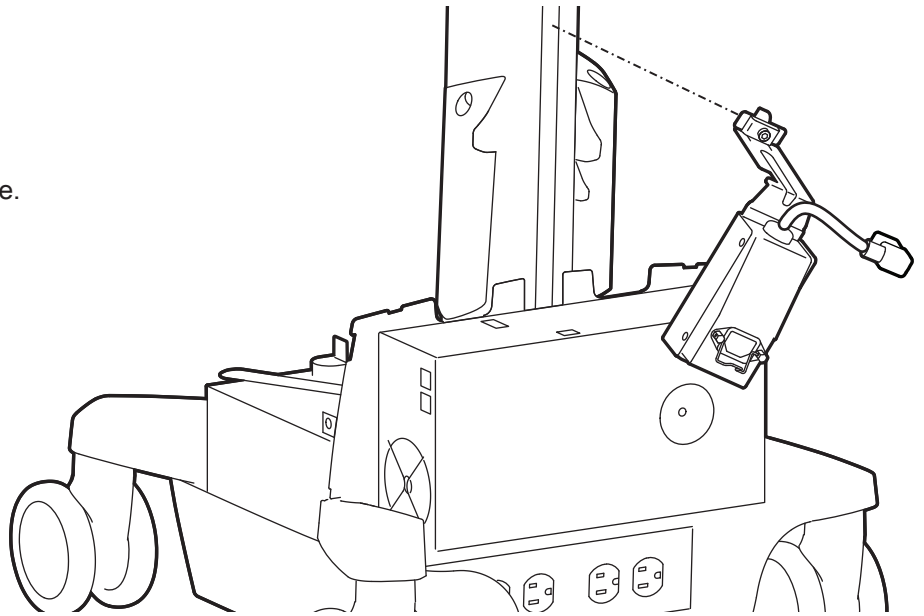
1 Insert bracket into T-slot at an angle.

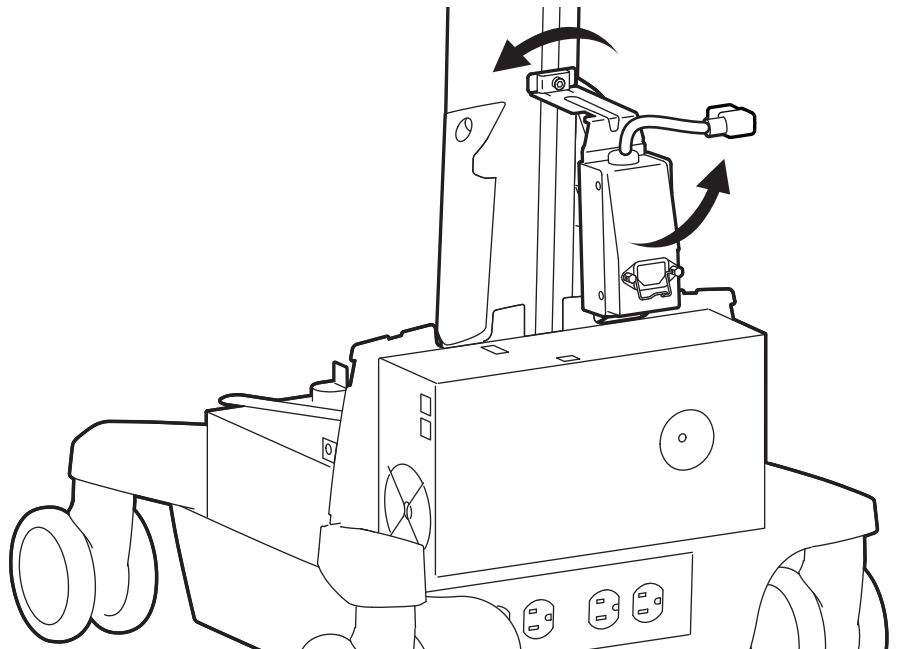
2 Rotate so bracket catches inside slot.

3 Slide down until it stops.

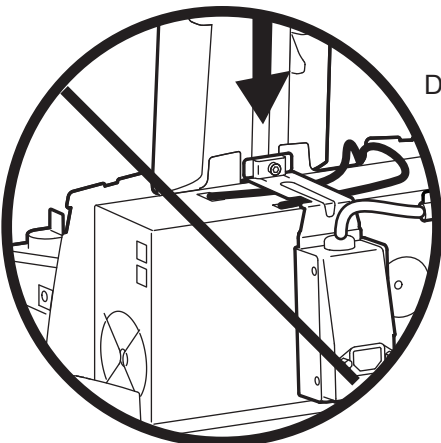
DO NOT pinch cables.

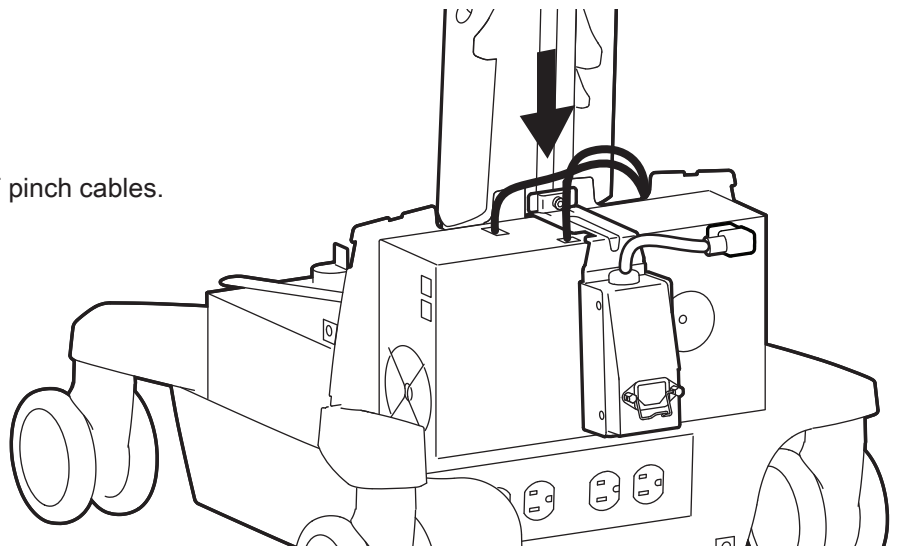
14

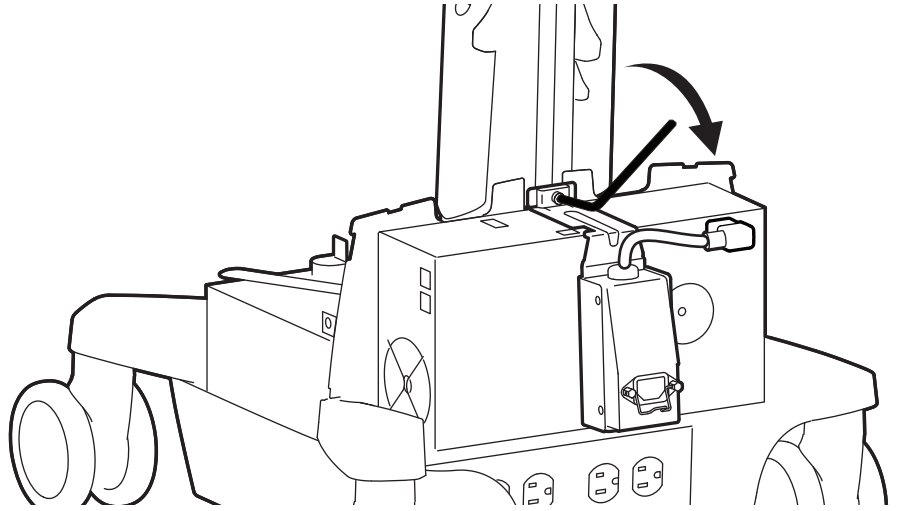Tighten to secure.




15

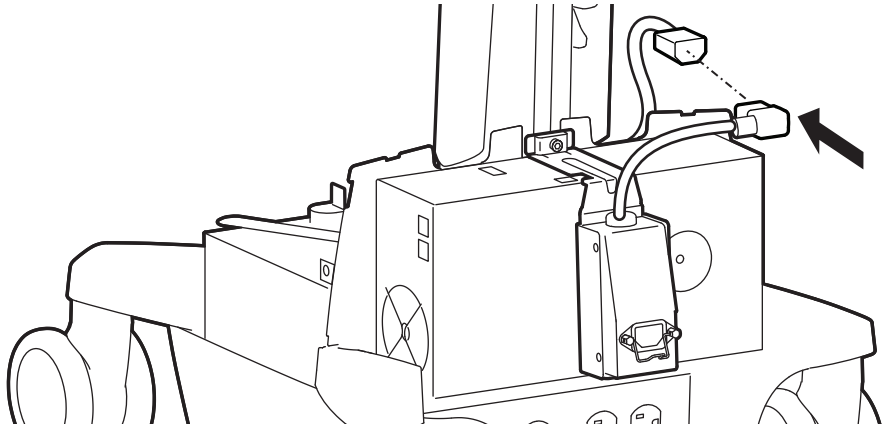Plug in power box.



16

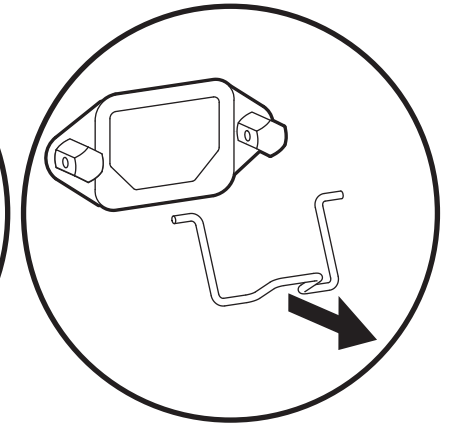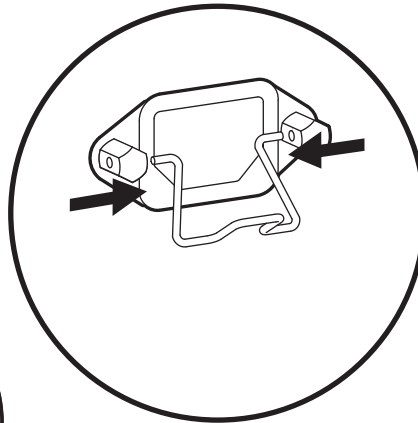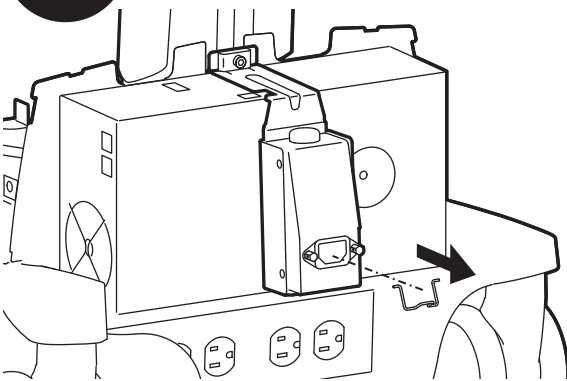Remove retainer.

17

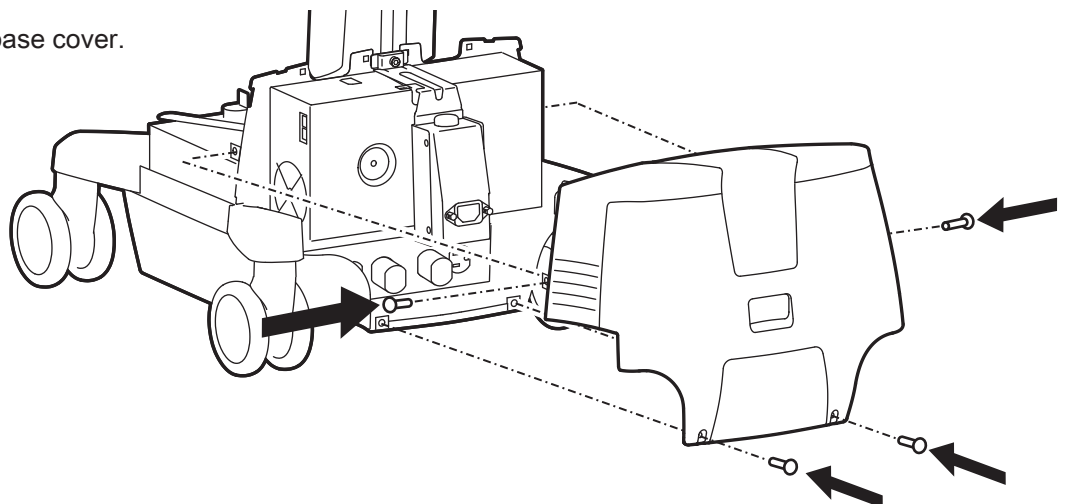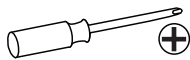
Attach new rear base cover.



**18**

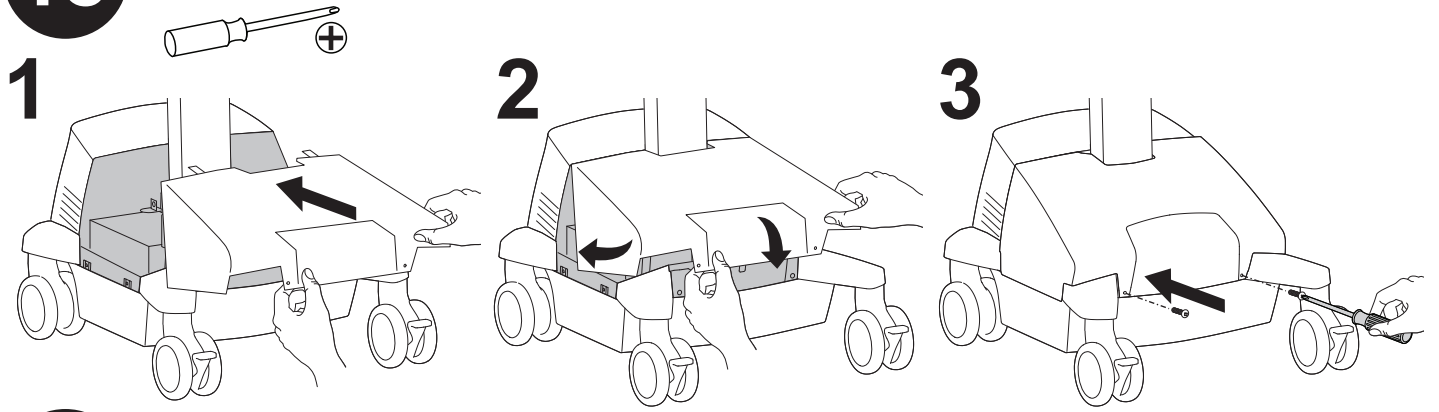Attach front base cover.



**19**

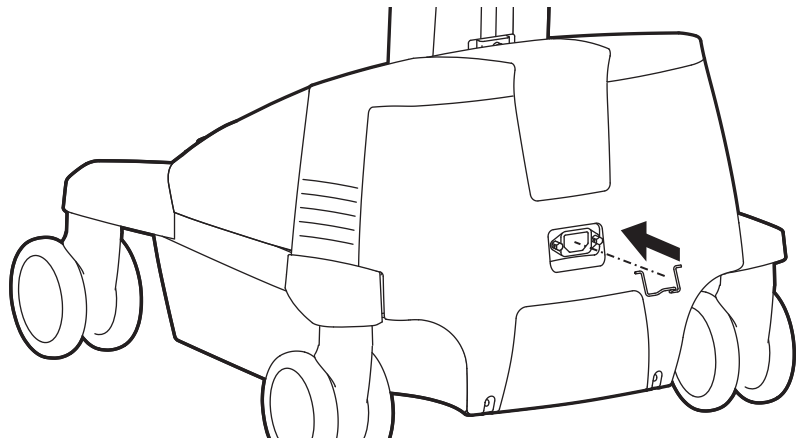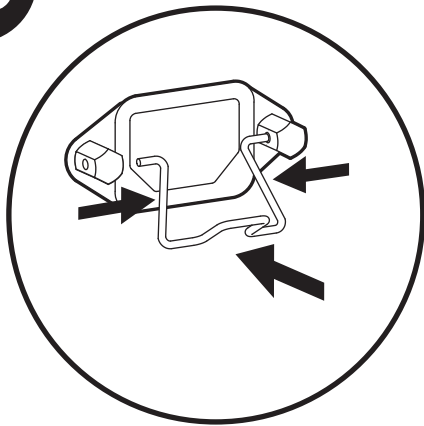Replace retainer.


**20**

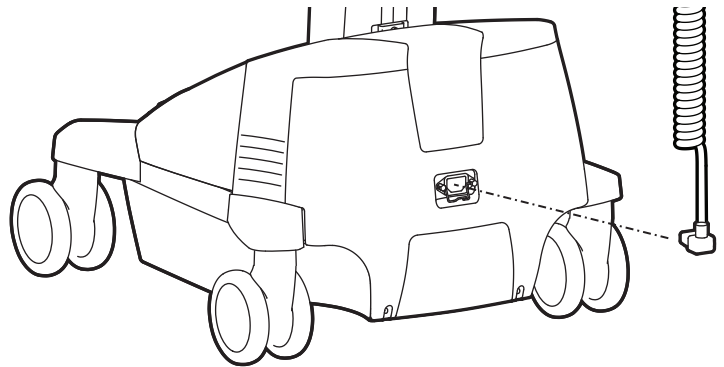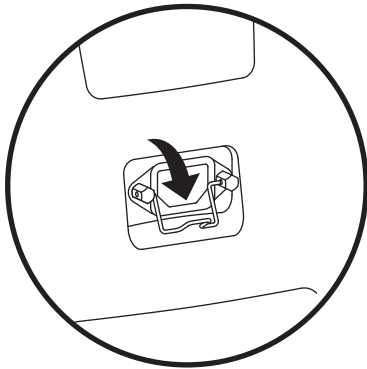Flip retainer down, hang power cord on hook and plug into base.



**21**

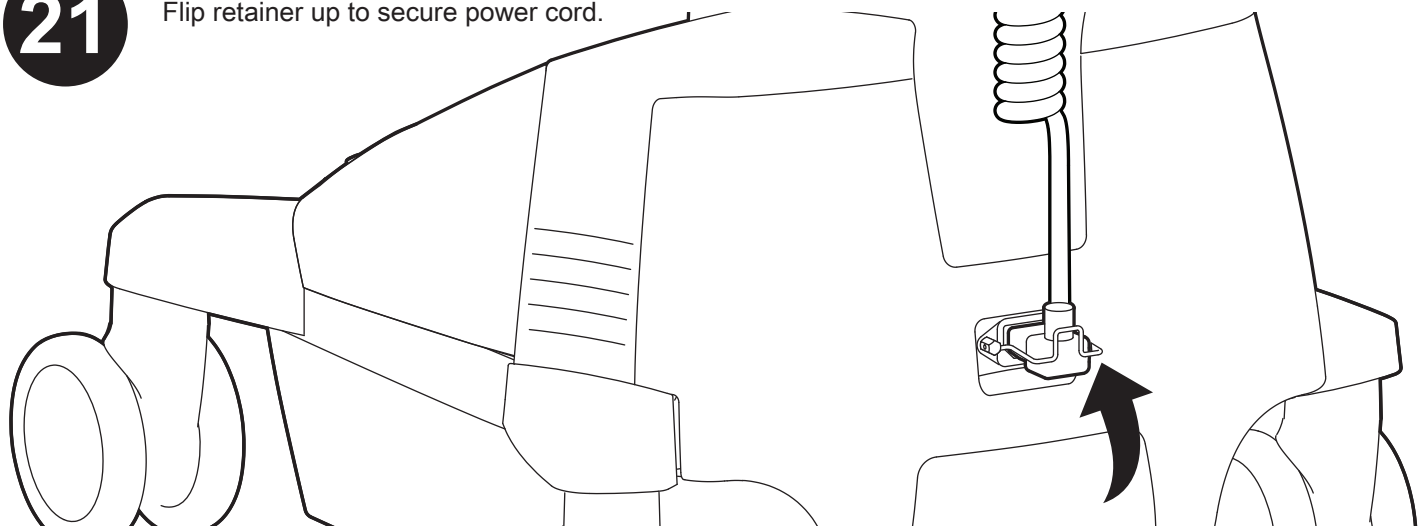Flip retainer up to secure power cord.



**22**

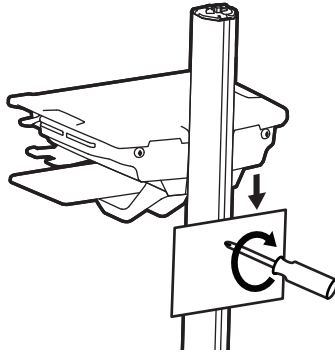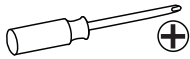Attach all T-slot mounted equipment.





**23**

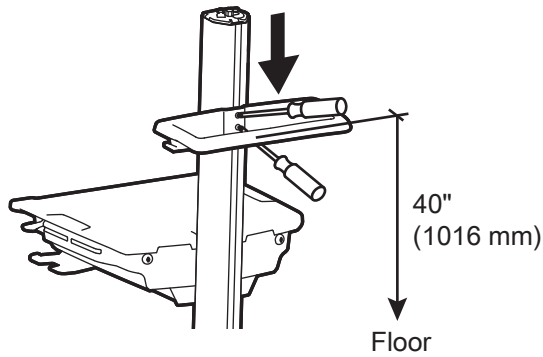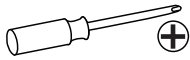Slide handle down T-slot and tighten screws.


**24**

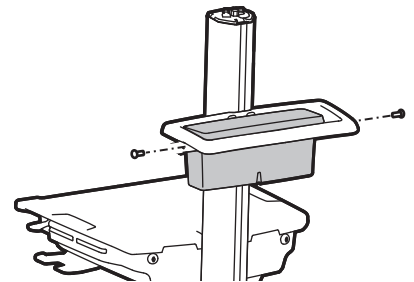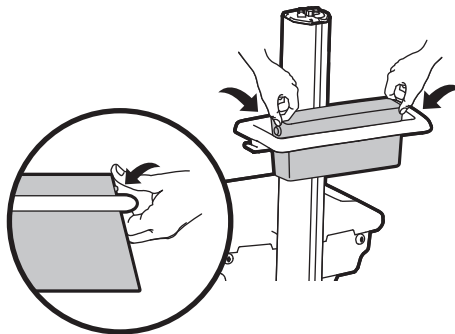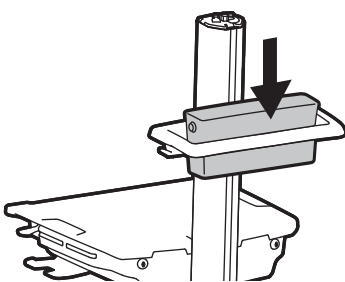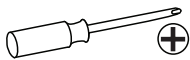Attach basket.




**25**

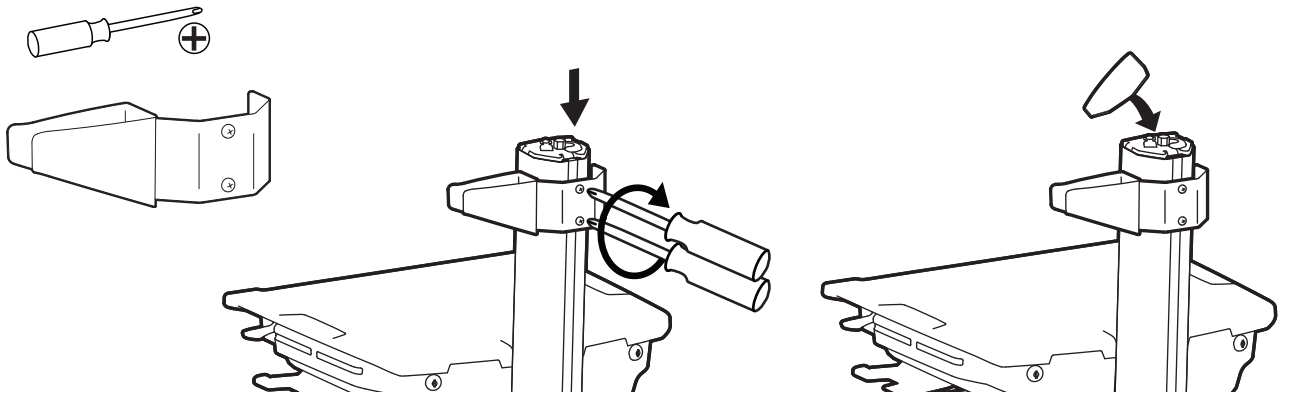
Attach scanner holder and top cover.

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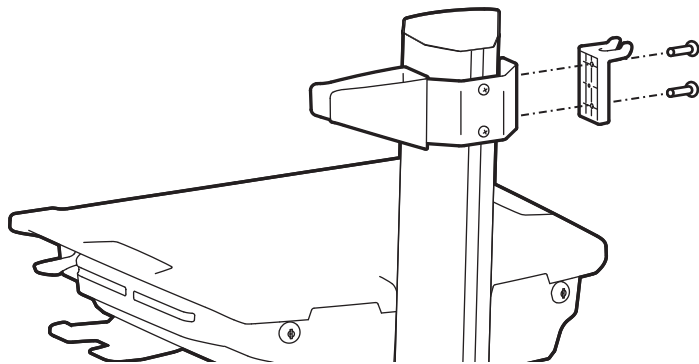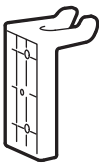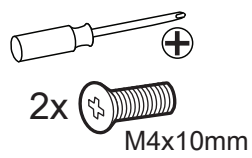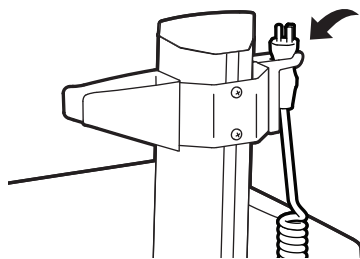© 2021 Ergotron, Inc. All rights reserved. StyleView™ is a registered trademark of Ergotron, Inc.

While Ergotron, Inc. makes every effort to provide accurate and complete information on the installation and use of its products, it will not be held liable for any editorial errors or omissions (including those made in the process of translation from English to another language), or for incidental, special or consequential damages of any nature resulting from furnishing this instruction and performance of equipment in connection with this instruction. Ergotron, Inc. reserves the right to make changes in the product design and/or product documentation without notification to its users. For the most current product information, or to know if this document is available in languages other than those herein, please contact Ergotron. No part of this publication may be reproduced, stored in a retrieval system, or transmitted in any form or by any means, electronic, mechanical, photocopying, recording or otherwise without the prior written consent of Ergotron, Inc., 1181 Trapp Road, Eagan, Minnesota, 55121, USA Patents Pending and Patented U.S. & Foreign. Ergotron is a registered trademark of Ergotron, Inc.